


HYALACE


BEST FOR YOUR SKIN

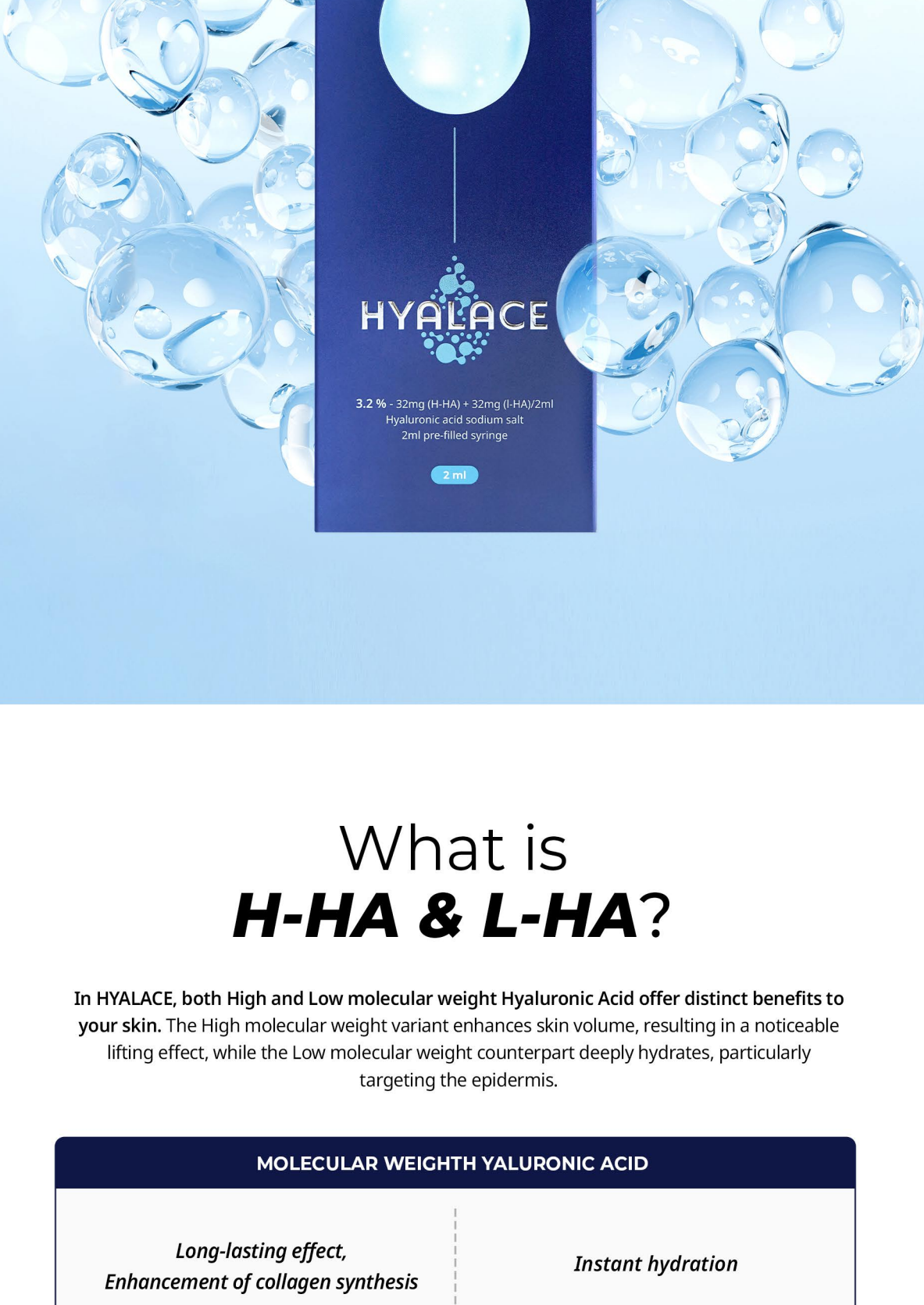

What is H-HA & L-HA?


In HYALACE, both High and Low molecular weight Hyaluronic Acid offer distinct benefits to your skin. The High molecular weight variant enhances skin volume, resulting in a noticeable lifting effect, while the Low molecular weight counterpart deeply hydrates, particularly targeting the epidermis.

MOLECULAR WEIGHT HYALURONIC ACID

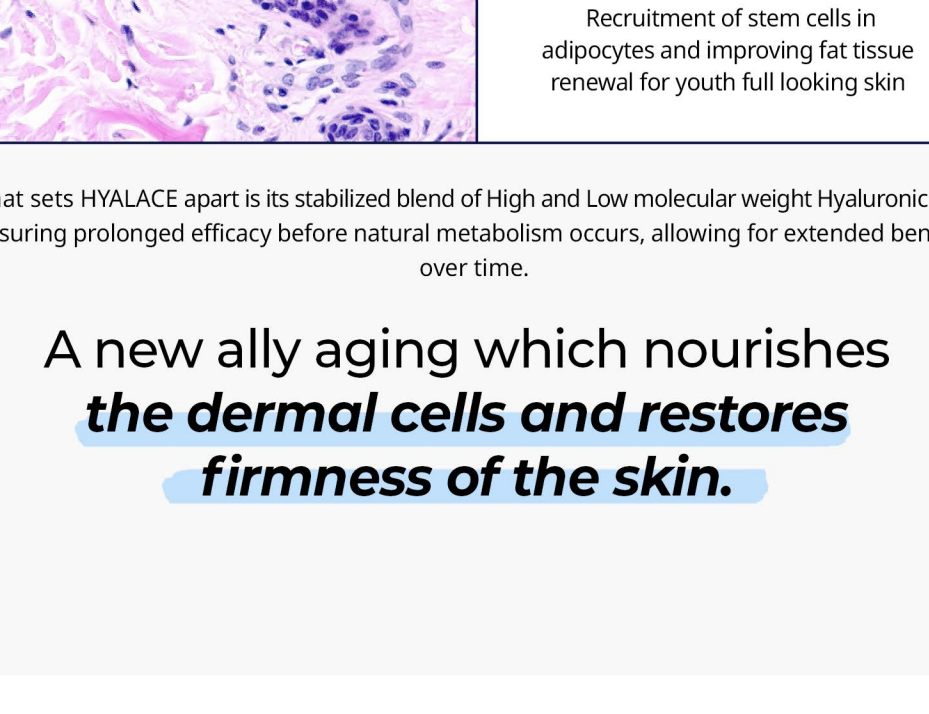

Furthermore, these two components synergize to amplify the production of type I and type III collagen in Fibroblasts, and type IV and VII collagen in Keratinocytes. Additionally, the treatment promotes elastin synthesis.

How does HYALACE work?

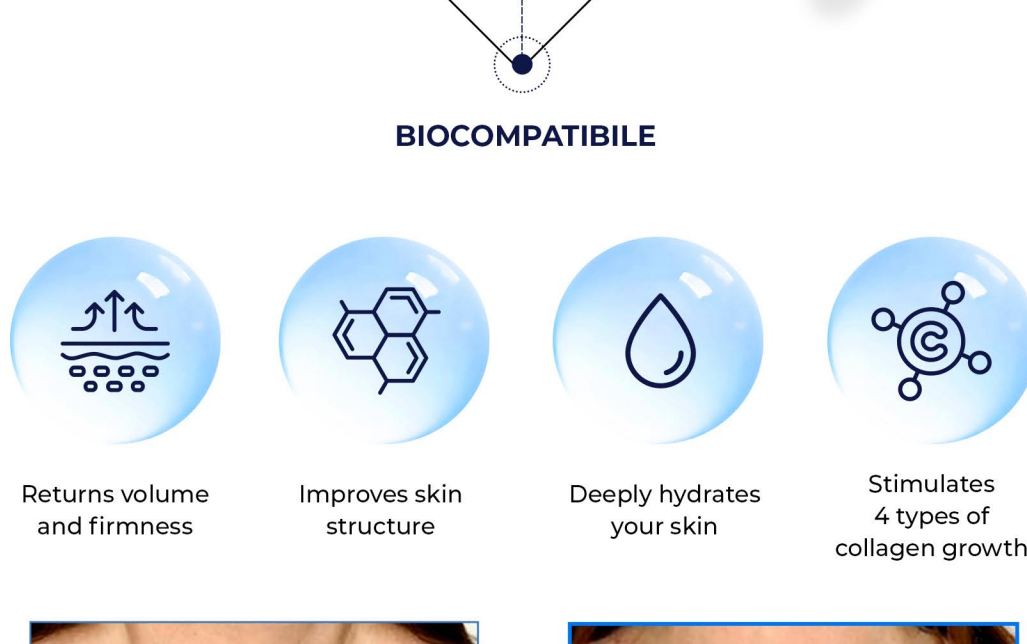

ACTIVATING KERATINOCYTES

Supporting Keratinocyte viability and promoting wound healing


STIMULATING FIBROBLASTS

Recruitment of stem cells in adipocytes and improving fat tissue renewal for youth full looking skin

What sets HYALACE apart is its stabilized blend of High and Low molecular weight Hyaluronic Acid, ensuring prolonged efficacy before natural metabolism occurs, allowing for extended benefits over time.

A new ally aging which nourishes the dermal cells and restores firmness of the skin.

Benefits of HYALACE

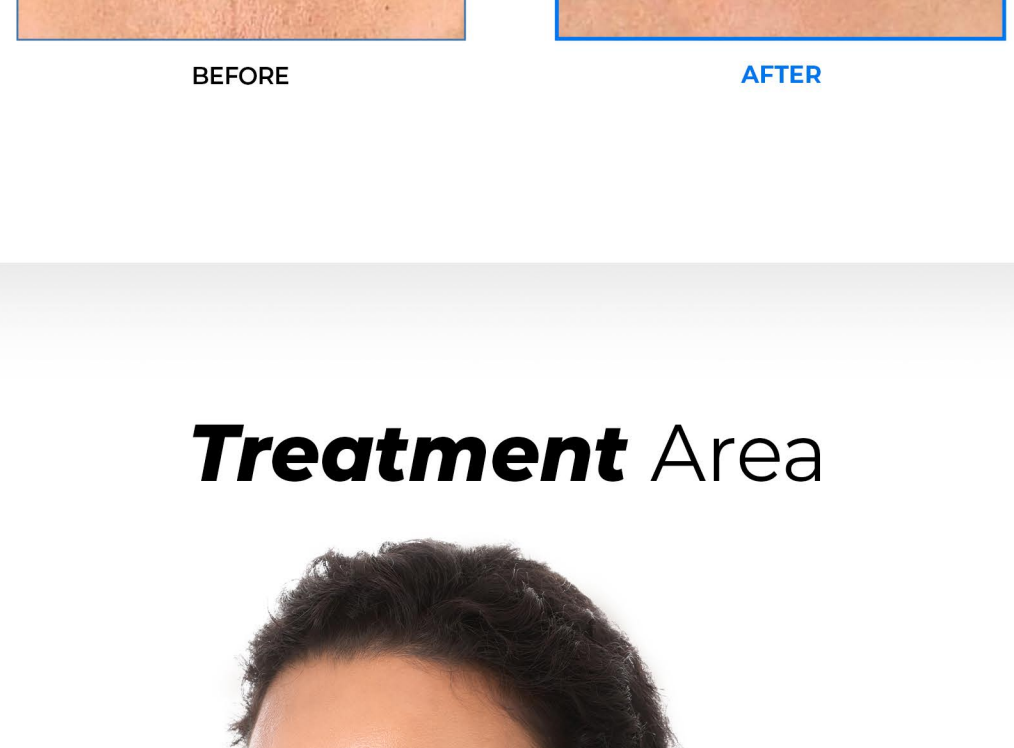

- INNOVATIVE**
- EFFECTIVE**
- BIOCOMPATIBLE**
- SECURE**

- Returns volume and firmness
- Improves skin structure
- Deeply hydrates your skin
- Stimulates 4 types of collagen growth



BEFORE

AFTER

Treatment Area



5 Point BAP technique

What is special about it?



5 POINT BAP


Unlike Skinboosters or Rejuran, HYALACE only needs 5 points per check for treatment of the whole face!

The 5 Bio Aesthetic Points (BAP) are specially selected areas of the face that have no large blood vessels or nerve branches, thereby minimizing the risk of bruising and improving safety while allowing for maximal diffusion of HYALACE's hybrid HA complex across the entire face. With only 5 point per check, discomfort and downtime is minimal to none.

FOR NECK REJUVENATION

Neck HYALACE treatment is excellent for targeting wrinkled or sagging neck skin.

Neck anti-aging benefits are stimulated from deep within the skin. 10 points along the neck are specially chosen to avoid vital structures such as blood vessels. The treatment experience is similar to HYALACE for the face.

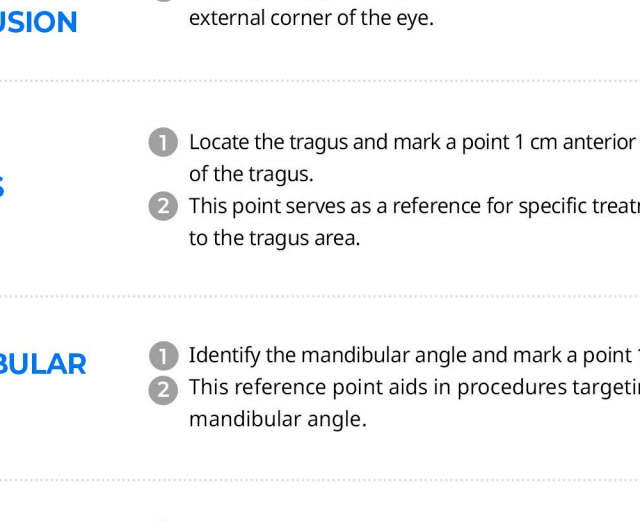


INSTRUCTION

PRE-TREATMENT INSTRUCTIONS

The injection process for HYALACE is a meticulously designed and clinically tested procedure aimed at achieving maximum efficiency and client comfort. The protocol involves the use of pre-filled syringes and follows the Bolus technique for precise application.

WORK PROTOCOL

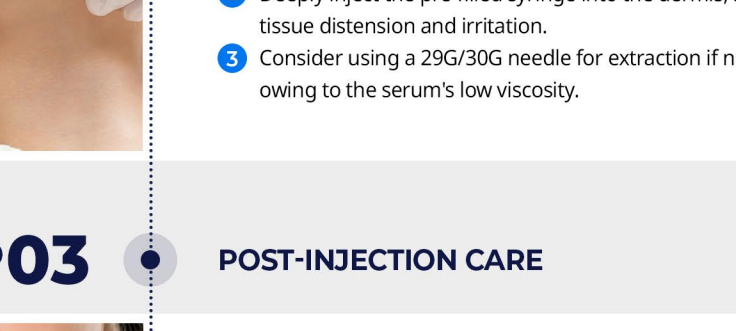

| | |
|----------------------------|---|
| Anesthesia | Local anesthetic cream |
| Techniques of Introduction | Bolus |
| Needle Type | Needle (29-30G x 13 mm) |
| Depth | 1-2 mm (superficial subcutaneous layer) |
| Angle of Inclination | 0-45° |
| Dose per Injection | 0.2 mL |

STEP 01


PREPARE SKIN FOR THE PROCEDURE



- Ensure a clean and sterile environment.
- Gather necessary equipment: pre-filled syringes, 29G needles, and alcohol swabs.
- Anesthetize the skin with anesthetic cream in preparation for treatment.
- Remove the anesthetic cream and clean with chlorhexidine the injection area.


STEP 02

ADMINISTRATION

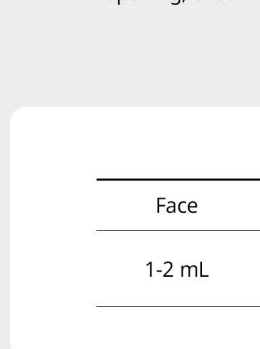

MAPPING

- Plan injection points to support progressive diffusion through sub-dermal patterns.

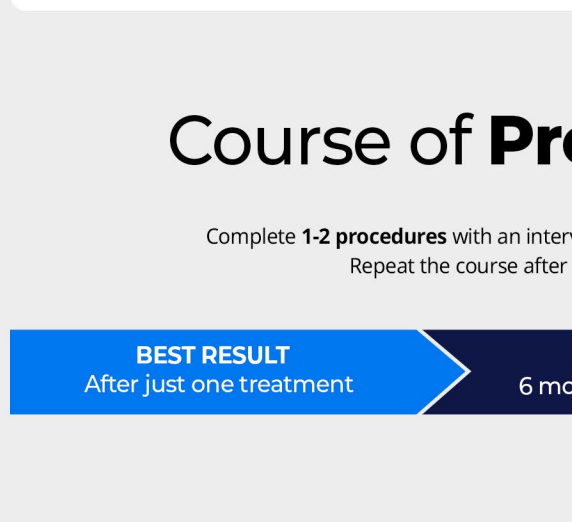

- ZYGOMATIC PROTRUSION
- TRAGUS
- MANDIBULAR ANGLE
- CHIN
- NASAL BASE

ZYGOMATIC PROTRUSION

- Identify the zygomatic protrusion at least 2 cm away from the external corner of the eye.

TRAGUS

- Locate the tragus and mark a point 1 cm anterior to the bottom of the tragus.
- This point serves as a reference for specific treatments related to the tragus area.

MANDIBULAR ANGLE

- Identify the mandibular angle and mark a point 1 cm above it.
- This reference point aids in procedures targeting the mandibular angle.

CHIN

- Draw a vertical line in the center of the chin.
- Draw a perpendicular line one-third from the top of the vertical line.
- From the point of intersection, move 1.5 cm towards the oral commissures.
- This precise marking assists in determining specific treatment areas on the chin.

NASAL BASE

- Draw a line connecting the nostril and tragus.
- Draw a perpendicular line starting from the pupil.
- Locate the injection point at the intersection of the 2 lines.




INJECTION

- Employ the Bolus technique for controlled and smooth release of Hyalace serum.
- Deeply inject the pre-filled syringe into the dermis, minimizing tissue distension and irritation.
- Consider using a 29G/30G needle for extraction if needed, owing to the serum's low viscosity.

STEP 03

POST-INJECTION CARE

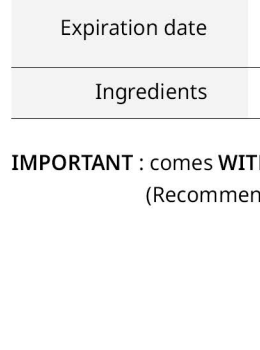

- Advise patients on aftercare, including avoiding excessive sun exposure, makeup, or strenuous activities.
- Provide any necessary post-injection instructions or prescriptions.

NOTE | HYALACE syringes are sterile and should be used before their expiration date, without mixing with other substances, and administered only to healthy skin to maintain safety and efficacy. The product, designed for single use to prevent contamination, must be stored properly, used immediately upon opening, ensuring that its packaging integrity and storage conditions are meticulously observed.

AMOUNT OF PRODUCT

| | | | | |
|--------|--------|----------------|------------|-----------------------|
| Face | Neck | Under the Eyes | Body | Hands |
| 1-2 mL | 1-3 mL | 0.5-1 mL | Up to 5 mL | 4 mL (for both hands) |

Course of Procedures

Complete 1-2 procedures with an interval of 60 days for Best result. Repeat the course after 6-12 months.

BEST RESULT
After just one treatment

ADDITIONAL PROCEDURE
6 months later for enhanced benefits

Warning

- After approximately 30 minutes, you should be able to return to your normal 'light' activities, including driving.
- It is recommended that you avoid wearing make-up for 24 hours and that you take a day off from strenuous physical exercise, such as visiting the gym or playing your favorite sport.
- Avoid saunas or a steamy bath, as well as intense sun or UV light, for three to four days.

Contraindications

Contraindications include patients with bleeding disorders, immunocompromised disorders and those with unrealistic expectations.

Specification

| | |
|-----------------|--|
| Product name | HYALACE |
| Packing unit | 2 mL * 1 Syringe, Instruction for use |
| Storage | Store at room temperature(1-25°C), protect from sunlight |
| Expiration date | 24 months from date of manufacture. Check the box for the expiration date. |
| Ingredients | Water, Sodium Hyaluronate, Sodium Chloride. |

IMPORTANT : comes WITHOUT A NEEDLE. The needle must be purchased separately. (Recommended needle size: 29G / 30G)