



Tesoro

Body Filler

Tesoro
Body Filler
MAX
100 mg/mL
60mL

Tesoro
Body Filler
100 mg/mL
60mL

Tesoro Body

Premium Body Filler

Ultimate body contouring with natural volume and long-lasting results.
Now available in two formats: 100 mL (50 mL * 2) syringe and 60 mL vial.

Mechanism

Tesoro Body is formulated using advanced HENM technology, offering greater density and elasticity than conventional fillers. The cross-linked hyaluronic acid with BDDE slows degradation and provides extended volume retention. Its high viscoelasticity ensures natural movement, shape



Convenient Vial & Syringe Formats

Tesoro Body is available in two convenient formats: a 50 mL pre-filled syringe for precise application, and a 60 mL vial for flexible dosing and multi-area treatments.

Benefit | 02

Non-invasive and Immediate Results

Unlike plastic surgery, Tesoro Body provides immediate aesthetic improvements with a simple injection procedure and a short recovery time.

Benefit | 03

High Durability with BDDE

The inclusion of BDDE cross-links Hyaluronic Acid, ensuring the maintenance of shape and providing long-lasting durability for the treated areas.

Benefit | 04

Safety and Purity

Made from pure Hyaluronic Acid and protected from exposure to air or air bubbles, Tesoro Body guarantees a safe and hygienic treatment experience, minimizing any risk of contamination.

Benefit | 05

Unlike plastic surgery, Tesoro Body provides immediate aesthetic improvements with a simple injection procedure and a short recovery time.



Safety and Purity

Made from pure Hyaluronic Acid and protected from exposure to air or air bubbles, Tesoro Bio

guarantees a safe and hygienic treatment experience, minimizing any risk of contamination



SHOULDERS

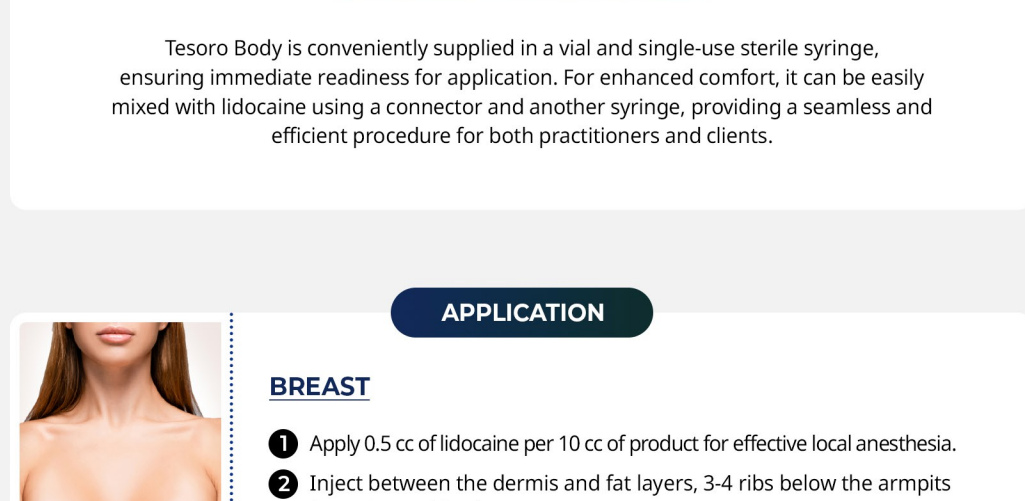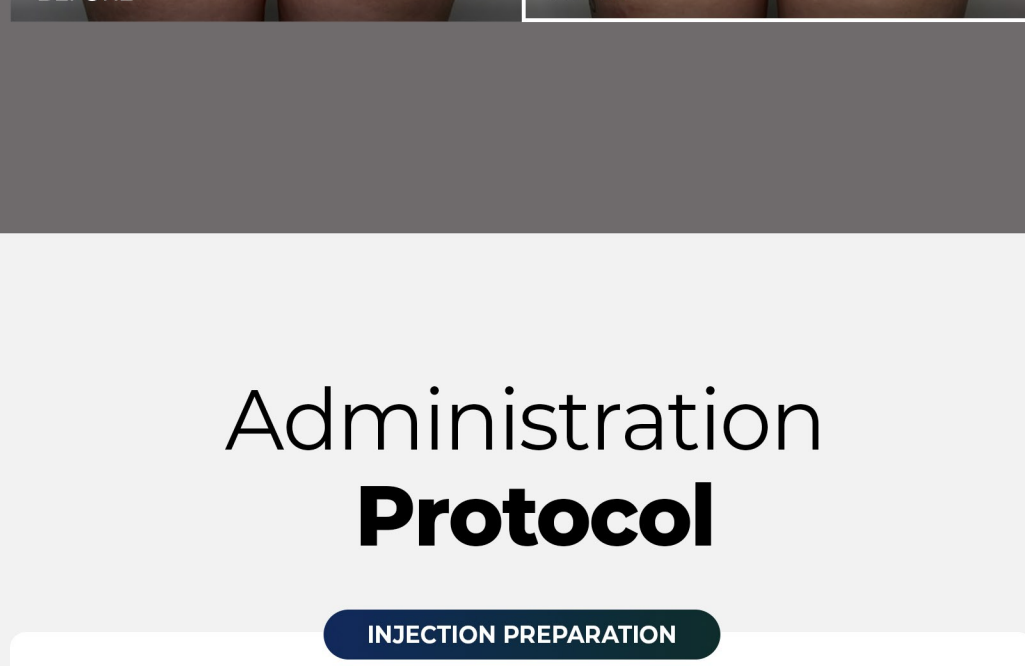

1. Apply 0.5 cc of lidocaine per 10 cc of product for effective local anesthesia.
2. Inject in the subcutaneous layer using the fan-shaped method.

FEMALE GENITALS

FEMALE GENITALS

vaginal entrance.

- 
- ### MALE GENITALS
- 1 Apply 0.5 cc of lidocaine per 10 cc of product for effective local anesthesia.

RECOVERY & LONGEVITY

Recovery time may vary based on individual factors but typically ranges from **4 to 15 days**.

- For long-lasting results, a touch-up session is recommended **2-3 months after** the initial treatment. This may extend the effects for **up to 18-36 months**.

- Mild swelling is normal and may persist for 5-7 days

- For single-use only. Do not re-sterilize.
- Do not use if packaging is damaged or opened.
- Inform patient of the potential risk of infection.

- Maintain a minimum interval of 6 months between treatments.
- ## Contraindications
- Avoid use in areas containing residual filler from previous treatments.
 - Do not inject into tendons, muscles, bones, or blood vessels.
 - Not recommended for periorcular areas or glabellar contouring.
 - Contraindicated in individuals with known hypersensitivity to hyaluronic acid.
 - Do not use on inflamed or infected skin.
 - Avoid overuse or exceeding the recommended dosage.
 - Not suitable for pregnant or lactating women.

Product		
Package	●●●●● (650-750 Pa)	●●●●● (650-750 Pa)
Storage	HA 20mg/mL	HA 20mg/mL

Expiration date	Up to 18 months	Up to 18 months
Usage	Store at a controlled room temperature (1–30°C), protected from direct sunlight and freezing.	Store at a controlled room temperature (1–30°C), protected from direct sunlight and freezing.
Composition	36 months from the date of manufacture. The expiry date is marked on each package.	36 months from the date of manufacture. The expiry date is marked on each package.