

# B-esta Body Filler

50ml




## Alternative to invasive procedures. For aesthetic body correction.

The filler is designed to correct : breasts, buttocks, lower legs, calf muscles and other parts of the body. Correction after unsuccessful invasive procedures, filling missing volumes that cause aesthetic discomfort to the patient. The most relevant area is the legs, knees and buttocks and arms. With age, the skin tends to lose elasticity and "sag". The B-esta50 Body HA Filler is ideal for these problems.

B-esta50 Body HA Filler is used to restore the volume and shape of certain areas of the body, for example : breasts, lower legs, buttocks, as well as to eliminate irregularities on the skin surface caused, for example, by liposuction. The filler provides a quick and long-lasting effect without surgery, avoiding a number of complications such as scarring.

Filler B-esta50 Body HA Filler is a pure gel based on stabilized Hyaluronic Acid, purified by a special method, which reduces the number of allergic reactions.

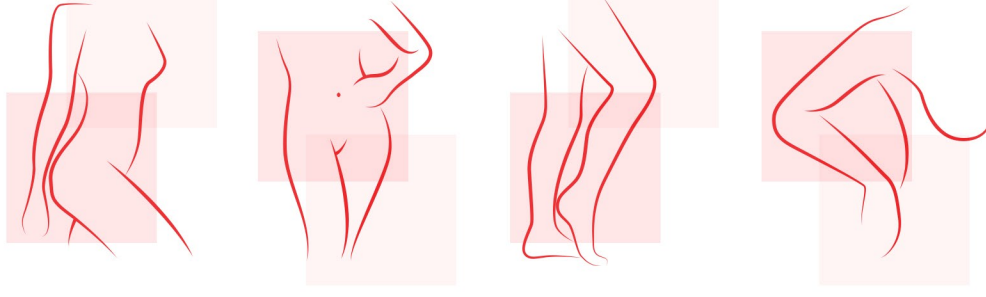

## Efficiency B-esta50 Body HA Filler

A single application of the filler gives an effect lasting up to two years, since the gel is biocompatible with the human body, it is absorbed and excreted naturally. To ensure a more stable and long-lasting result, the correction can be repeated after a month.

The filler contains stabilized biocompatible hyaluronic acid, which is excreted naturally without causing side effects.

## Procedure

The procedure takes 30-40 minutes.

After the procedure, a specialist may prescribe antibiotics to avoid rejection of the injected product. When modeling the breast, the product is injected with caution, avoiding the mammary gland.

## Introduction technique

The injection technique is determined by a specialist, it depends on the place of correction. It is advisable to carry out the procedure in 2 stages, the time interval is 2 weeks.

## How quickly will the result be seen after the procedure?

The effect is noticeable immediately, like after contouring. The duration of the effect depends on many factors. Each organism is individual. Filler body shaping provides a natural increase in volume. The visible result depends on the amount of injected product, as well as on the condition of the patient's skin. If the patient does not have a sufficient volume of pvc, more product will be required.

Before starting the procedure, it is necessary to take a history, apply local anesthesia, and make a markup. Take a photo before the procedure. Filler is inserted using 18 / 22 G needles or 18 / 22 G cannulas.



## How to use

### STEP 1

Remove the filler  
with a 10ml syringe

### STEP 2

Puncture with  
18G / 22G needle

### STEP 3

Attach the  
18G / 22G cannula  
to the pre-filled  
10 ml syringe

### STEP 4

Inject B-esta50 Body  
HA Filler into the  
treated area

## Contraindications

- Tendency to hypertrophic and keloid scars
- Intolerance to gram-positive bacteria
- Active inflammatory or infectious processes
- Old or chronic skin conditions
- Undergoing anticoagulant therapy
- Allergy to Hyaluronic Acid
- Autoimmune diseases

## Structure

Sodium hyaluronate - 24mg/ml  
Lidocaine hydrochloride monohydrate - 3mg/ml

Sodium phosphate

Sodium chloride

Potassium phosphate monobasic



## Release form

Sterile 50ml bottle

For single use

It is important to record the date of the injection

## Storage rules

Do not freeze or expose to high temperatures. Store at room temperature (2-25 C).

## Note

If the box and ampoule are damaged, the product cannot be used.

Don't use the B-esta50 Body Ha Filler.